
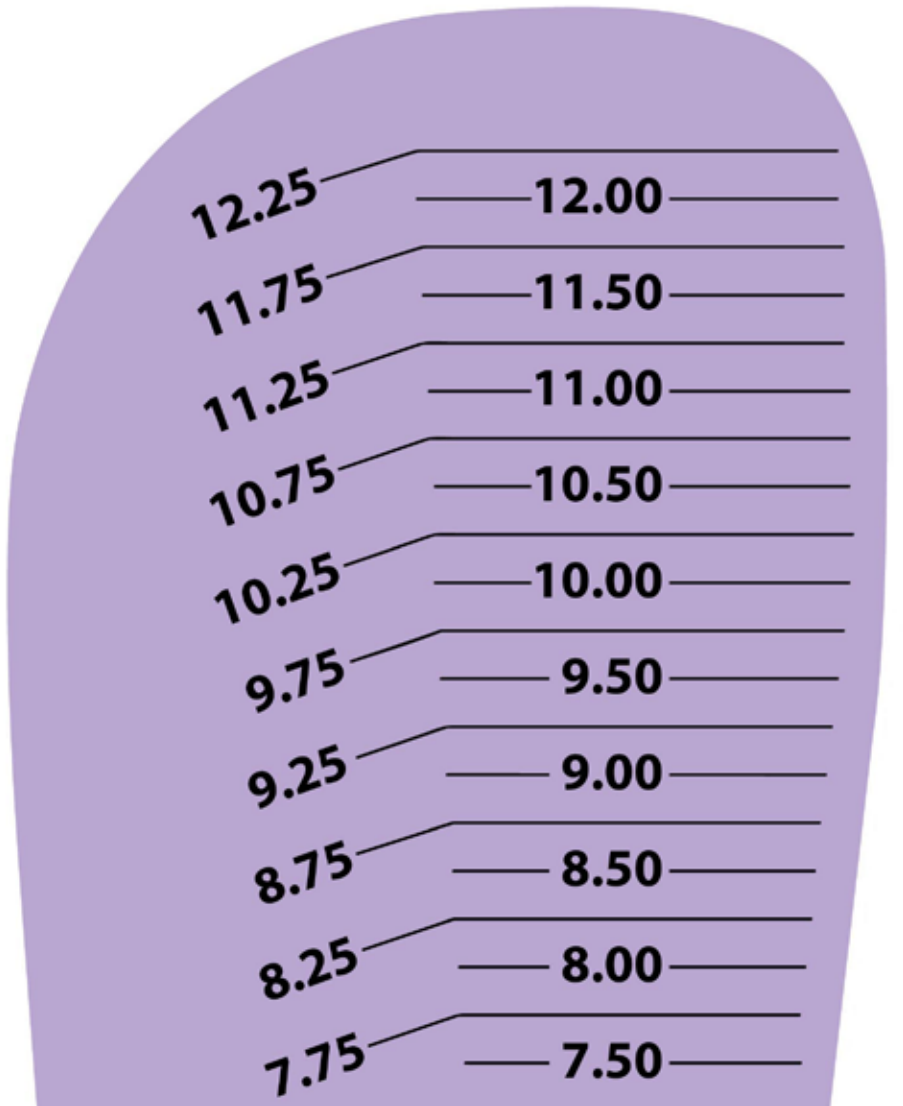


# Fast Fit<sup>®</sup> Sizing Jig | Printable Version

## Left Foot – Top

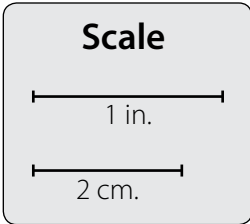

### Questions?

Contact our **Customer Service** team:  
800.848.7332 | [customerservice@dafo.com](mailto:customerservice@dafo.com)  
Monday–Friday, 5:30am–5:00pm (PST)

*Cut/fold along dotted line. Connect Bottom portion of Left Foot here.*

▼7.25

### 1 Print

To ensure accurate scaling and proper sizing, use the following printer settings:

1. With the Printable Sizing Jig pdf open in Adobe Reader, go to the File menu.
2. Click the Print sub-menu item.
3. Under Page Scaling, select None. (DO NOT select Fit to Printable Area or Fit to Page. If there's a percentage option, make sure it's set at 100%.)
4. Print.
5. After printing, verify that the scale on this sheet is accurate with a standard ruler.

### 2 Assemble

1. Follow the printing instructions (to the left). If the scaling is accurate, continue on.
2. Cut/fold along the dotted lines on the jigs' Top and Bottom portions.
3. Tape the Top and Bottom portions of the Left and Right sizing jigs at their corresponding dotted lines.

### 3 Size

1. Make sure the patient is wearing socks and have him/her stand on the jig. Align the outermost part of the heel and the inside of the foot with the designated lines on the jig.  
*\*See Figures 1 and 2 above.*
2. Choose the length that will allow ¼ to ½ in. (.64 to 1.3 cm.) for growth.
3. Unless the patient's foot is significantly narrow, we recommend ordering a Wide size.

# Fast Fit® Sizing Jig | Printable Version

## Left Foot – Bottom

Cut/fold along dotted line. Connect Top portion of Left Foot here.

▲7.50

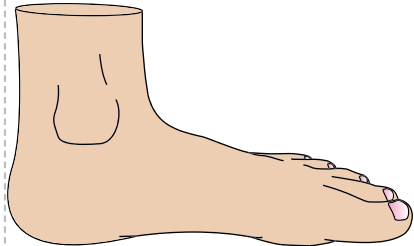


Figure 1: Outermost part of heel

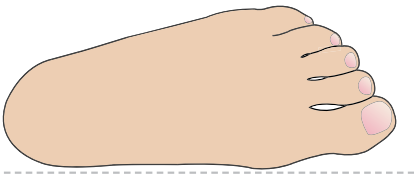


Figure 2: Inside of foot

### Sizes currently available

(0.25 in. increments—Wide and Narrow)

HotDog

PattiBob

4.00–12.25 in.

Chipmunk



Cricket

JumpStart Leap Frog

4.00–9.00 in.

JumpStart Bunny

JumpStart Kangaroo

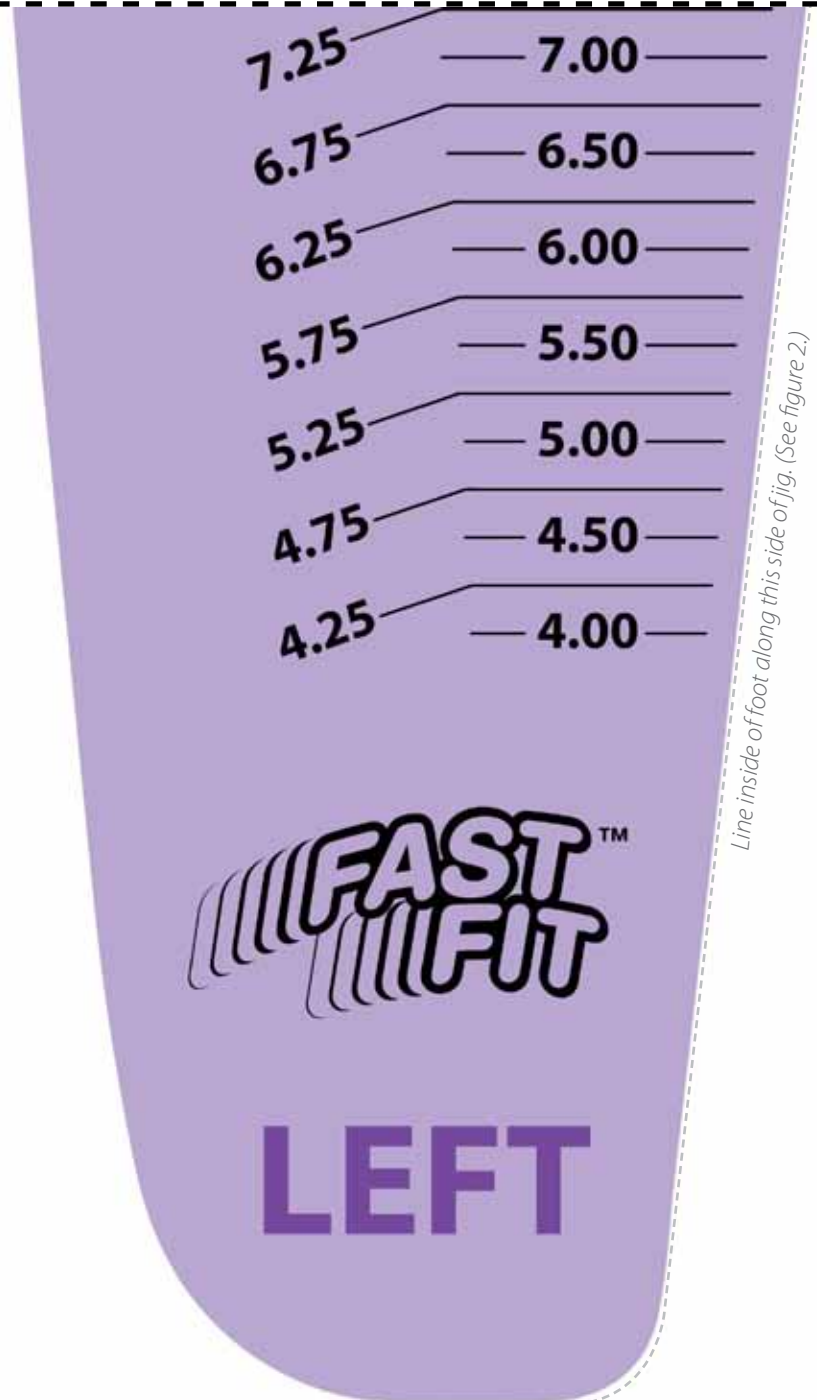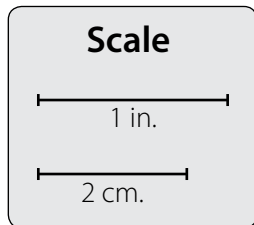



Line inside of foot along this side of jig. (See figure 2.)

Line outermost part of heel along this line. (See figure 1.)

Printable Sizing Jig — Left Bottom rev.02.1 (Oct 2012)

# Fast Fit<sup>®</sup> Sizing Jig | Printable Version

## Right Foot – Top

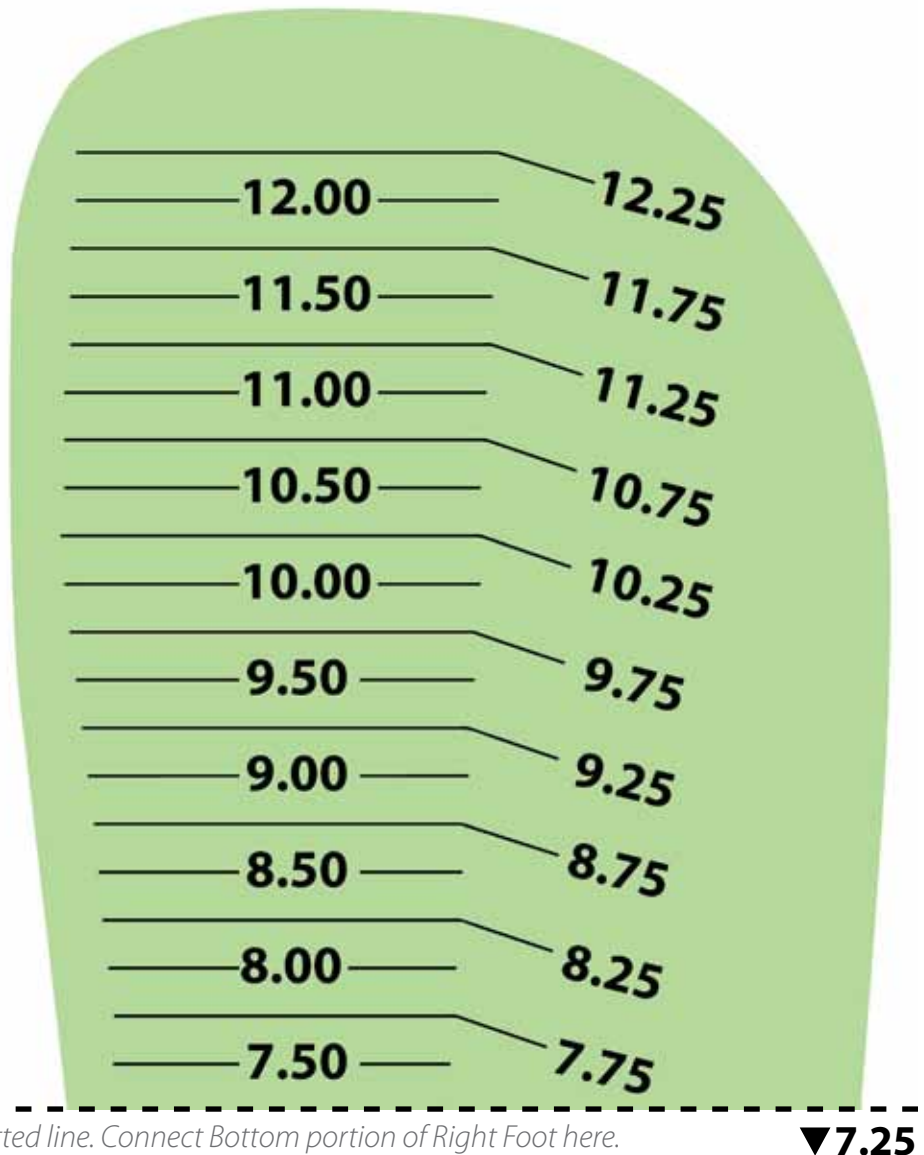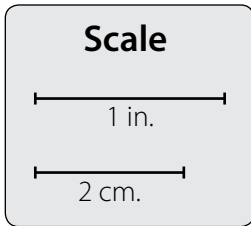

### Questions?

Contact our **Customer Service** team:  
800.848.7332 | [customerservice@dafo.com](mailto:customerservice@dafo.com)  
Monday–Friday, 5:30am–5:00pm (PST)

### 1 Print

To ensure accurate scaling and proper sizing, use the following printer settings:

1. With the Printable Sizing Jig pdf open in Adobe Reader, go to the File menu.
2. Click the Print sub-menu item.
3. Under Page Scaling, select None. (DO NOT select Fit to Printable Area or Fit to Page. If there's a percentage option, make sure it's set at 100%.)
4. Print.
5. After printing, verify that the scale on this sheet is accurate with a standard ruler.

### 2 Assemble

1. Follow the printing instructions (to the left). If the scaling is accurate, continue on.
2. Cut/fold along the dotted lines on the jigs' Top and Bottom portions.
3. Tape the Top and Bottom portions of the Left and Right sizing jigs at their corresponding dotted lines.

### 3 Size

1. Make sure the patient is wearing socks and have him/her stand on the jig. Align the outermost part of the heel and the inside of the foot with the designated lines on the jig.  
*\*See Figures 1 and 2 above.*
2. Choose the length that will allow ¼ to ½ in. (.64 to 1.3 cm.) for growth.
3. Unless the patient's foot is significantly narrow, we recommend ordering a Wide size.

# Fast Fit® Sizing Jig | Printable Version

## Right Foot – Bottom

Cut/fold along dotted line. Connect Top portion of Right Foot here.

▲7.50



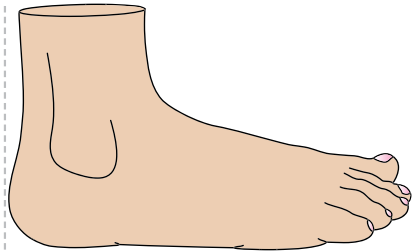


Figure 1: Outermost part of heel

Figure 2: Inside of foot

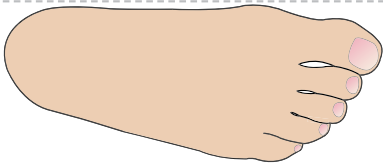

Line inside of foot along this side of jig. (See figure 2.)

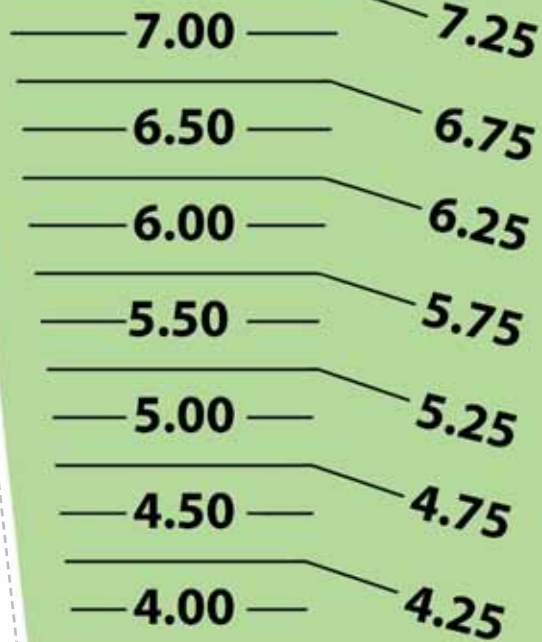

### Sizes currently available

(0.25 in. increments—Wide and Narrow)

HotDog

PattiBob

4.00–12.25 in.

Chipmunk

Cricket

JumpStart Leap Frog

4.00–9.00 in.


JumpStart Bunny

JumpStart Kangaroo


**FAST  
FIT™**

**RIGHT**

### Scale

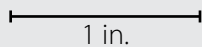

1 in.

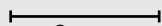

2 cm.

Line outermost part of heel along this line. (See figure 1.)

Printable Sizing Jig — Right Bottom rev.02.1 (Oct 2012)